


# Sternal Talon<sup>®</sup> Measurement Chart

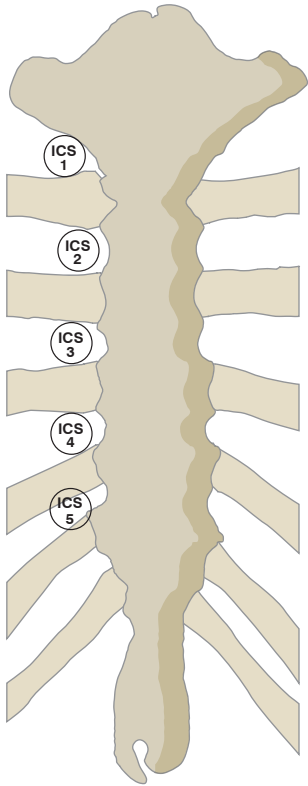

	Left Width (mm)	Right Width (mm)	Total Sternum Width (mm)	Sternum Depth (mm)	Talon Size	Stock #	Lot #
ICS 1	_____	+	= _____	_____	_____	_____	_____
ICS 2	_____	+	= _____	_____	_____	_____	_____
ICS 3	_____	+	= _____	_____	_____	_____	_____
ICS 4	_____	+	= _____	_____	_____	_____	_____
ICS 5	_____	+	= _____	_____	_____	_____	_____

		Total Sternum Width									
		XS (17-22mm)		S (23-31mm)		M (32-38mm)		L (39-45mm)		XL (46-52mm)	
		Single	Double	Single	Double	Single	Double	Single	Double	Single	Double
Sternum Depth	≤11mm	<b>S11XS</b> 24-008-11	<b>S11S</b> 24-009-11	<b>D11S</b> 24-019-11	<b>S11M</b> 24-010-11	<b>D11M</b> 24-020-11	<b>S11L</b> 24-011-11	<b>D11L</b> 24-021-11	N/A	N/A	
	12-14mm	<b>S14XS</b> 24-008-14	<b>S14S</b> 24-009-14	<b>D14S</b> 24-019-14	<b>S14M</b> 24-010-14	<b>D14M</b> 24-020-14	<b>S14L</b> 24-011-14	<b>D14L</b> 24-021-14	<b>S14XL</b> 24-012-14	<b>D14XL</b> 24-022-14	
	15-17mm	N/A	<b>S17S</b> 24-009-17	<b>D17S</b> 24-019-17	<b>S17M</b> 24-010-17	<b>D17M</b> 24-020-17	<b>S17L</b> 24-011-17	<b>D17L</b> 24-021-17	<b>S17XL</b> 24-012-17	<b>D17XL</b> 24-022-17	
	18-20mm	N/A	N/A	N/A	<b>S20M</b> 24-010-20	<b>D20M</b> 24-020-20	<b>S20L</b> 24-011-20	<b>D20L</b> 24-021-20	<b>S20XL</b> 24-012-20	<b>D20XL</b> 24-022-20	

\*Note: 1- If two adjacent intercostal spaces have a total width difference greater than 2mm, a double Talon may not be indicated.  
 2- If sternum thickness between left and right halves is not equal, the surgeon may choose to use larger/deeper device.  
 Care should be exercised to ensure device is in close proximity to inferior table of the sternum and poses no harm to vital structures.